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## 1 General Information

The new HEBU permanent filters are a further step towards more safety and economic efficiency by the preparation, respectively sterilization of medical products.

The HEBU permanent filters offer you the following advantages over the previous long term filters made from textile material:

- Number of sterilization cycles > 1200
- Fits all sterilizing container models
- High economic efficiency through minimizing usage costs
- Saves time due to the short amount of time needed to check the filter and prevention of a filter change

## 2 Intended use

A Permanent filter serves as a bacterial barrier and is a reusable product that can be used for all HEBU-sterilizing containers.

## 3 Usage / Handling

The handling, respectively the insertion of the permanent filter remains unchanged with regard to the previously used filter.

The permanent filters are only to be handled from trained, respectively schooled personnel in order to prevent damages.

The permanent filter is to be checked for damages before each and every sterilization cycle.

The filter holder must be pressed into place, until it clicks, so that it can be heard after the permanent filter is inserted. The permanent filters need no cleaning and can remain in the filter holder until the end of the utilization.

## 4 Material

PTFE (Teflon®)

The filtration procedure of various media with PTFE filters is already known for a long time and just recently made useful for the usage in sterilizing containers.

## 5 Testing / Inspection

An independent and accredited testing laboratory carried out tests and examinations as to the suitability of the permanent filters. The results are as follows:

- The used HEBU- PTFE permanent filters are suitable to prevent bacterial growth according to the requirements of DIN 58953 Part 6, Chapter 2.14.

## 6 Special Information

To guarantee flawless functioning of the permanent filters, please pay attention to the following information:

- ⚠ Never puncture the sterile filters!
- ⚠ Prevent the placement of sharp and pointed objects on the sterilizing containers
- ⚠ Prevent the penetration of bacteria through the spilling of dirt, cleaning or disinfection solutions
  - Penetration of contaminated external condensate (for example through the space
  - between the stack of hot containers)
  - Rubbing of dust and dirt in the filter by wiping the containers
  - Long term storage

## 7 Disposal

After usage, the permanent filters can be disposed of in the same customary way as hospitals do. No special measures are necessary.

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**Our vast range of products covers over 10 000 various instruments.  
They can be found in Internet or request our catalog.**

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HEBU medical GmbH
**HEBUmedical GmbH**