HF 9527

HF 9528

HF 9529

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Contents

1	Symbols	10
2	Introduction	10
3	Scope	10
3.1	Inteded Use	10
3.2	Contraindications.....	10
4	Construction	11
5	Warning.....	11
6	Handling	12
7	Preparation.....	12
7.1	Information on instrument preparation	12
7.2	Preparation at the place of use	12
8	Maintenance, inspection	12
9	Storage.....	12
10	Warranty / Repair	12
11	Manufacturer and servicing address.....	13

1 Symbols

Symbol	Definition
	CE marking
	Attention
	Manufacturer

2 Introduction

By purchasing this equipment trolley, you are now the owner of a high-quality product whose use and correct handling are described in the following. In order to minimize possible risks to patients and users, please observe these instructions carefully.

3 Scope

The use of equipment trolleys takes place in operating rooms.

Equipment trolleys are mainly used in operating rooms for the HF surgery.

The equipment trolley was designed especially for HF devices. On the surface of the table there are special holder for a better hold of the device. The equipment trolley also has a deposition shelf compartment for instruments.

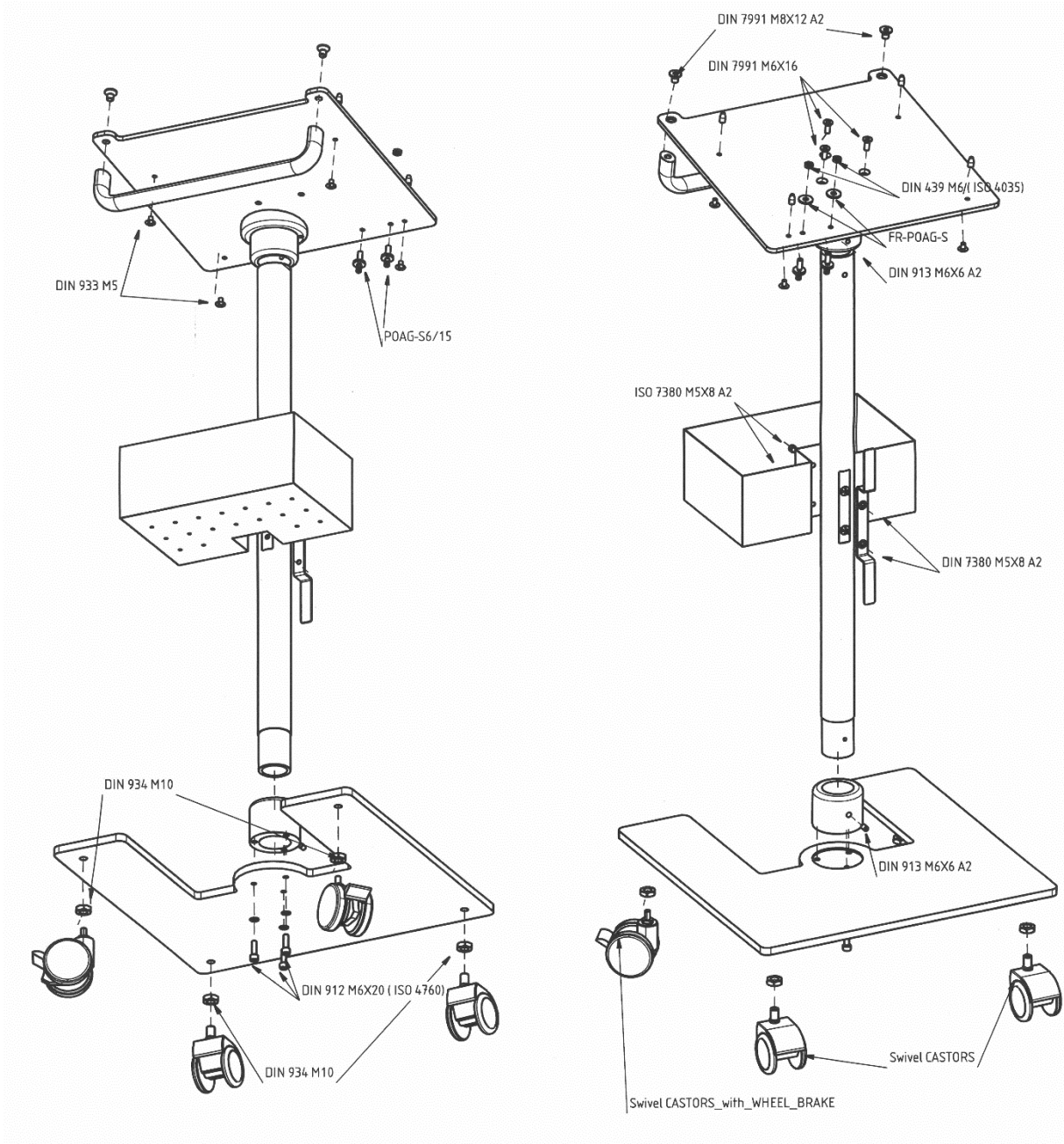
3.1 Inteded Use

Equipment trolley for electrosurgical units: A trolley used as a means of transport to transport one or more electromechanical devices. This mobile trolley has one or more horizontal shelves in which the devices are positioned.

3.2 Contraindications

There are no known contraindications.

4 Construction

5 Warning

The equipment trolley must be cleaned and disinfected before first use

Please observe the additional information enclosed with the products

Inspect the equipment trolley for damage after every period of use.

A damaged trolley may not be used.

6 Handling

The product may only be used by appropriately trained and qualified personnel.

After each use, the equipment trolley must be checked for damaged. It is necessary to ensure that the HF device has a secure hold on the equipment trolley.

7 Preparation

7.1 Information on instrument preparation

- Use cleaning and/or disinfection agents with a pH-value within 9-10.
- Please observe manufacturer instructions regarding dosage, exposure time and renewal of solutions.
- Do not use hard brushes or coarse abrasive cleaners.

7.2 Preparation at the place of use

For cleaning the equipment trolley a wipe disinfection is recommended. We recommend using lint-free, soft cloths for manual cleaning. The cleaning agent should have no fixing effect.

8 Maintenance, inspection

Carry out the functional check mentioned above. Products with stains, which are bent, no longer function or which are otherwise damaged must be segregated!

9 Storage

Store products in a dry, clean and dust-free environment at moderate temperatures from 5°C to 40°C. Protect the product from the effects of the sun's rays and artificial light.

10 Warranty / Repair

Our products are manufactured from high-grade materials and carefully checked prior to dispatch. However, even if used properly in accordance with their intended purpose they are subject to a greater or lesser degree of wear depending on their intensity of use.

This wear is technically induced and unavoidable.

Should faults occur independently of wear, please contact our customer services. Defective products should no longer be used.

They must undergo the complete preparatory treatment process before being returned.

11 Manufacturer and servicing address

HEBUmedical GmbH
Badstraße 8
78532 Tuttlingen / Germany
Tel. +49 7461 94 71 - 0
Fax +49 7461 94 71 - 22
eMail: service@HEBUmedical.de
Web: www.HEBUmedical.de

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**Our vast range of products covers over 10 000 various instruments.
They can be found in Internet or request our catalog.**

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