Laryngoskop / Laryngoscope

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Seite 3

Operating Manual

Page 7

DEUTSCH

ENGLISH

Inhalt

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Contents

1	General Information	8
2	Intended use	8
3	Use of new laryngoscopes	8
3.1	Sterilization of blades with bulbs	8
3.2	Sterilization of cold light blades.....	8
3.3	Sterilization of Handles	8
4	Aftersales service address	9

1 General Information

All laryngoscopes are manufactured through a very careful production process with regular quality checks. Observe the following care and sterilization procedures to ensure a long working life.

2 Intended use

A laryngoscope is a device that is used to introduce the tracheal tube during intubation. It is a long thin rod-like device, typically angled at the distal end designed to give a good directional control during routine or difficult oral intubations. This is a reusable device.

3 Use of new laryngoscopes

Prior to first use, all laryngoscopes must be cleaned and sterilized.

3.1 Sterilization of blades with bulbs

All blades are either autoclavable up to 134 ° C (3 bar g) or can be sterilized in hot air up to 140°C without contact damage. The bulbs and light carriers must be removed prior to sterilization. Sterilize bulbs and carriers separately in gas or a disinfective solution.

3.2 Sterilization of cold light blades

All cold light blades are autoclavable up to 134° C (3 bar g). To enhance the working life of the in-built glass fibres, gas sterilization or disinfection in solution is recommended. Do not clean cold light blades in an ultrasonic cleaner. Never apply them to flash-autoclavers, hot air sterilization, or chemicals.

3.3 Sterilization of Handles

The battery handles should neither be autoclaved nor sterilized in hot air. Only sterilize in gas or a disinfective solution. For this procedure, do not forget to take out the batteries.

To avoid corrosion or insufficient contact use only non-leaking Alkaline batteries. Concerning handles with cold light equipment only gas sterilization or disinfection in solution are applicable. Remove the bulbs before sterilization.

Attention!

Improper care and handling or non-intended use can lead to premature wear and tear of laryngoscopes.

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**Our vast range of products covers over 10 000 various instruments.
They can be found in Internet or request our catalog.**

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HEBU medical GmbH
HEBUmedical GmbH