HB 9150-12
HB 9150-24

HEBU insta-hold

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DEUTSCH

Operating Manual
Page 7

ENGLISH

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FRANÇAIS

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ESPAÑOL

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ITALIANO

HB 9150-12
HB 9150-24

HEBU insta-hold

Inhalt

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HEBU insta-hold

Contents

1	Delivery programme.....	8
2	Scope	8
2.1	Inteded Use	8
2.2	Contraindications.....	8
3	Operating	8
4	Cleaning and sterilisation.....	9
5	Storage.....	10
6	Warranty.....	10
7	Environmental protection	10
8	Manufacturer and servicing address.....	10

1 Delivery programme

HB 9150- 12	3 Pcs HEBU insta-hold /package incl. 5 fixation buttons L*B: 480mm (19“) * 305mm (12”)
HB 9150-24	3 Pcs HEBU insta-hold /package incl. 5 fixation buttons L*B: 480mm (19“) * 610mm (24”)
HB 9150-99	Backup- Buttons, 5 pcs/package

2 Scope

The HEBU Insta Hold is used in operations.

2.1 Inteded Use

Instrument Holder Mat: The HEBU Insta-Hold Mat is used to fix instruments during an operation. Almost all instruments in the operating environment can be fixed on any surface. It is a reusable product.

2.2 Contraindications

There are no known contraindications.

3 Operating

Congratulations: You are now the owner of the **HEBU insta-hold**, a precision engineered medical product from the company **HEBUmedical**.

This revolutionary development facilitates the work of any surgeon by providing a whole new standard of safety and convenience.

The **HEBU insta-hold** mat allows practically every type of instrument used in the operating environment to be firmly secured on different surfaces without the risk of magnetization (see Fig. 3).

The **HEBU insta-hold** mat can be cut to any optional shape (see Fig.1). If required you can fold a lateral pocket with fixation buttons (Fig.2) for a save positioning of the instruments (see Fig.3)

Fig. 1

To use, place the sterile mat with the smooth side upwards on the patient next to the operating area

Fig. 3

Fig. 2

The **HEBU insta-hold** fixing mat consists of silicone, avoid damage of sharp objects.

4 Cleaning and sterilisation

Please observe the following instructions:

- The fixing mat can be cleaned mechanically as well as manually. Always use a cleaning agent with pH value <10.
- The mechanical cleaning is possible with any alkaline agents.
- We recommend the use of soft brushes or cloths for the manual cleaning.
- Pay attention that the fixing mat is completely unrolled during the mechanical cleaning.
- Before first time sterilizing, peel off the protective film backing from the smooth side of the fixing mat! (Fig. 4)
- The fixing mat can be autoclaved at 134°C after an efficient cleaning
- For sterilization the cleaned mat can be convoluted

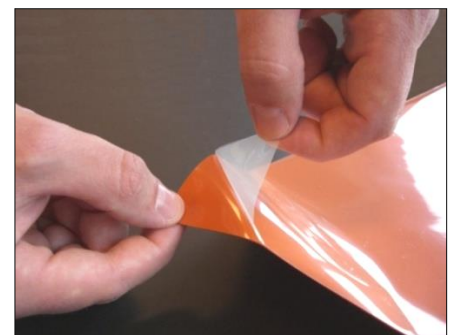
Fig. 4

5 Storage

Store products in a dry, clean and dust-free environment at moderate temperatures from 5°C to 40°C. Protect from the effects of the sun's rays and artificial light.

6 Warranty

We provide a guarantee for **HEBU insta-hold** in accordance with statutory and country-specific legislation (proof of purchase by invoice or delivery note). The warranty period is two years. Damage caused by incorrect handling or normal wear and tear is not covered by the warranty. In case of warranty claims, please enclose a copy of the invoice or delivery note with the device.

7 Environmental protection

Any packaging materials and disused fixing mat must be sent for environmentally friendly recycling or disposal. For more information on this subject, consult your specialist dealer.

8 Manufacturer and servicing address

HEBUmedical GmbH
Badstraße 8
78532 Tuttlingen / Germany
Tel. +49 7461 94 71 - 0
Fax +49 7461 94 71 - 22
eMail: service@HEBUmedical.de
Web: www.HEBUmedical.de

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