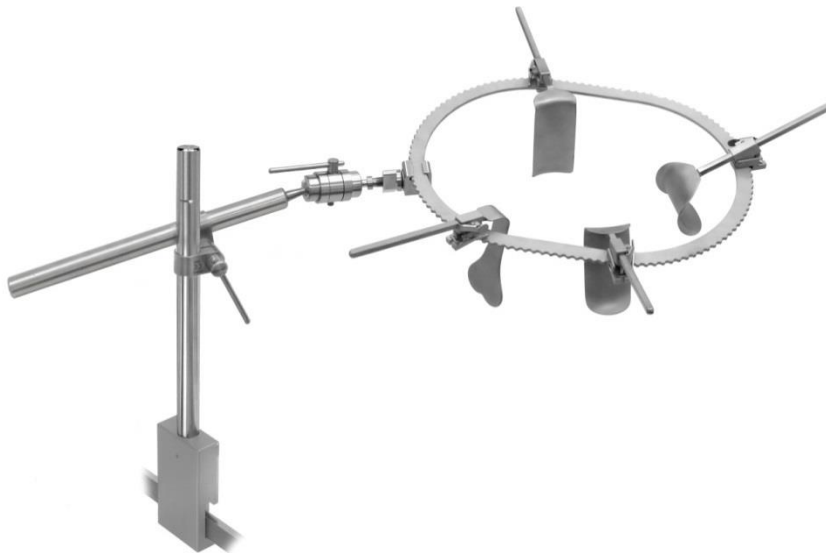
***HB 3337-00 bis HB 3337-10***

# HEBU U.R.S.

**Universal Retraktor System**

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Seite 3

**Operating Manual**

Page 9

**DEUTSCH**

**ENGLISH**

## ***HB 3337-00 bis HB 3337-10***

### **Inhalt**

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## ***HB 3337-00 to HB 3337-10***

### **Content**

<b>1</b>	<b>Safety and application remarks</b>	<b>10</b>
<b>2</b>	<b>Application</b>	<b>10</b>
2.1	Intended use	10
2.2	Contraindications	10
2.3	Overview of some single Components	10
<b>3</b>	<b>Reprocessing</b>	<b>11</b>
3.1	Point of Use	11
3.2	Preparation for Cleaning	11
3.3	Cleaning: Automated	11
3.4	Cleaning: Manual	12
3.5	Drying	12
3.6	Sterilization	12
3.7	Inspection and Function Testing	12
<b>4</b>	<b>Maintenance</b>	<b>12</b>
<b>5</b>	<b>Containment and Transportation</b>	<b>12</b>
<b>6</b>	<b>Instructions for Use</b>	<b>13</b>
6.1	Assemble & Disassemble	13
<b>7</b>	<b>Aftersales Service Address</b>	<b>13</b>
<b>8</b>	<b>Guarantee of Conformity</b>	<b>14</b>

## 1 Safety and application remarks

Supplied non-sterile. Must be cleaned and sterilized before use

Functional life demonstrated to 5 years only

## 2 Application

The HEBU U.R.S. is exclusively designed for surgical field excavations of various surgical procedures. These include, but are not limited to, bariatric, urological and gynaecological procedures.

### 2.1 Intended use

The HEBU U.R.S articulated arm system is a product for temporarily holding and positioning instruments, endoscopes and other products during a surgical procedure. It consists of a base, adjustable articulated arms and height-adjustable columns to adjust the instruments to the desired position. It is a reusable product.

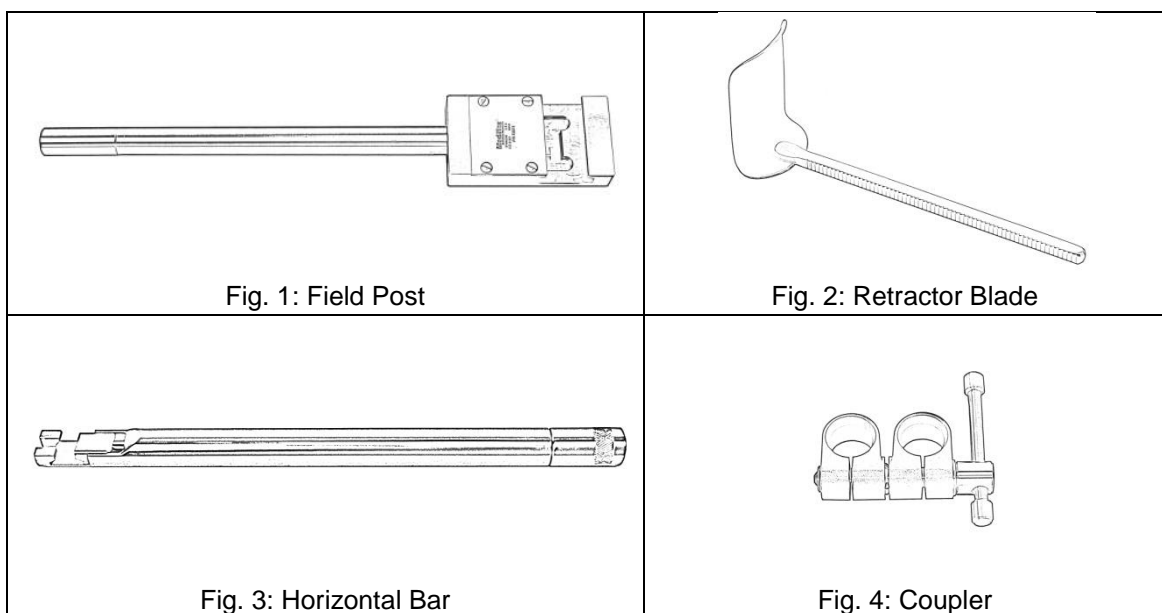
### 2.2 Contraindications

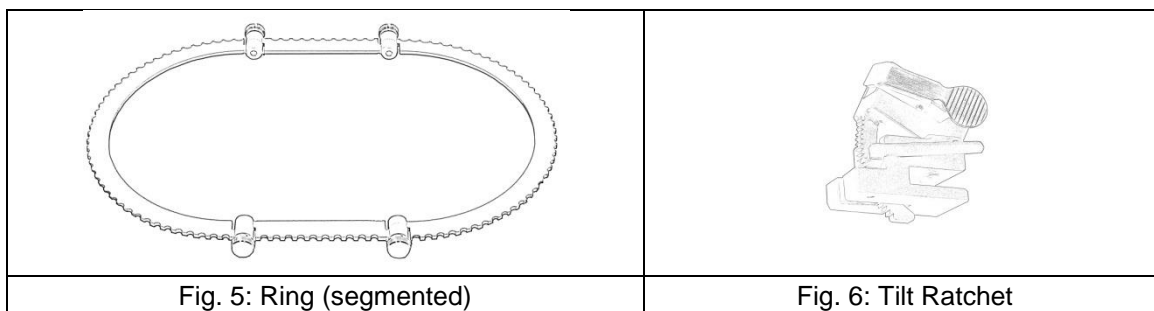
No contraindications are known.

### 2.3 Overview of some single Components

HEBU U.R.S. is made up of well over 100 different items, which can be assembled individually subject to its requirements.

Some single components are shown below to clarify the basic steps of assembling and disassembling the retracting system.

### 3 Reprocessing

#### 3.1 Point of Use

- Remove excess soil with disposable wipe.
- Moisten with distilled water, or enzymatic foam or presoak.

#### 3.2 Preparation for Cleaning

Separate components that are still assembled. Examples:

- Remove Couplers from Field Posts and Horizontal Bars.
- Loosen Couplers to separate the 2 halves.
- Remove Blades from Ratchets and Tilt Ratchets.
- Remove Rings from Claspers and Horizontal Bars
- Segmented Rings should be taken apart. The screws are captivated and should stay in place when the segments are separated.

#### 3.3 Cleaning: Automated

- Separate all components before Cleaning
- HEBU U.R.S. and its Components can be cleaned in Washer- Decontaminators using a pH neutral or pH<10 detergent such as MediClean forte by "Dr. Weigert". (If an alkaline detergent is used, alkaline streaking may result in Stainless Steel Surfaces. The use of a neutralizer may prevent this.)
- Cold water prewash should not exceed 40° C
- High Flow / Low Pressure detergent washes should be at temperatures of at least 90° C but at a maximum of 98° C.

**Other automated systems may be used if hospital protocol is established for reusable surgical instruments**

### 3.4 Cleaning: Manual

- Separate components as described above
- Open Couplers and Tilt Ratchets to expose all surfaces.
- Rinse thoroughly under running tap water at 27° C.
- Immerse components in a detergent. Prepare according to the manufacturer's instruction.
- Using a hand-held, soft bristled brush, clean all surfaces.
- The square opening in Ratchets and Tilt Ratchets should be cleaned with a 10 mm bottle brush. Smaller Opening should be cleaned with a 5 mm bottle brush.
- Visually inspect each component for any signs of remaining debris. Repeat steps above if necessary.

### 3.5 Drying

- Air Dry, or use a soft non-abrasive, non-linting towel.
- Compressed air may also be used to dry the device.

### 3.6 Sterilization

Pre-Vac Steam Autoclave: Minimum of 4 minutes at 132° C

### 3.7 Inspection and Function Testing

- Check to be sure moving parts, including tightening devices move freely with no binding.
- Visually inspect for burs, cracks and other damage.
- If instrument has been reassembled for testing after cleaning, it must be disassembled for sterilization.

## 4 Maintenance

We recommend regular lubrication of all threaded and hinged components. If any component is damaged, replacement components are available.

## 5 Containment and Transportation

- Place heavier items below lighter objects.

## 6 Instructions for Use

### 6.1 Assemble & Disassemble

- 1 Attach Field Post (Fig. 1) to OR Table Rail.
- 2 Insert a Horizontal Bar (Fig. 3) into Coupler (Fig. 4).
- 3 Attach Coupler to Field Post so the Horizontal bar is away from the surgeon and the jaws of the Horizontal Bar are above the surgical field.
- 4 Open the jaws of the Horizontal Bar and attach the appropriate Ring (Fig. 5) to the jaws. Tighten the jaws of the Horizontal Bar until ring is rigid.
- 5 Loosen the Coupler in order to change the position of the ring and position the ring over the operative sight. When in the appropriate position, retighten.
- 6 Select the Retractor Blade (Fig. 2) to be used and insert into a Ratchet (Fig. 6) in such a way that the teeth of the Blade engage the ratchet mechanism.
- 7 Insert the blades into the incision in the usual manner; pull back on the ratchet and attach the ratchet to the ring.
- 8 Adjust the retraction distance and the tilt manually. Tension of the tissue being retracted will hold the ratchet on the ring.
- 9 The Ratchet may be moved along the ring in order to reposition it.
- 10 Continue to place Blades around the operative site in order to maximize surgeon visibility.
- 11 To remove at the end of the procedure, reverse the above steps.

## 7 Aftersales Service Address

HEBUmedical GmbH  
Badstraße 8  
78532 Tuttlingen / Germany  
Tel. +49 7461 94 71 - 0  
Fax +49 7461 94 71 - 22  
eMail [service@HEBUmedical.de](mailto:service@HEBUmedical.de)  
Web: [www.HEBUmedical.com](http://www.HEBUmedical.com)

## 8 Guarantee of Conformity

This is to certify that:

HEBUmedical GmbH guarantees that the individual products marketed and sold as the *HEBU Universal Retractor System* are manufactured of the same materials as and are dimensionally equal to *Bookwalter™* branded Table Mounted Retractor Systems marketed by Codman.

HEBUmedical further guarantees that all *HEBU U.R.S.* components will interchange seamlessly with all *Bookwalter* Devices marketed by Codman.

If any HEBU U.R.S. component proves not to fit with Codman branded *Bookwalter* products, the HEBUmedical device may be returned for 100% credit.

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**Our vast range of products covers over 10 000 various instruments.  
They can be found in Internet or request our catalog.**

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Thomas Butsch, CEO HEBUmedical GmbH